Biosigma s.r.l.

a Dominique Dutscher Company

Via Valletta, 6 | 30010 Cantarana di Cona (VE), Italy | Tel. ++39 0426 302224 (r.a.) |

Fax ++39 0426 302228 | SMS ++39 348 4077376 |

E-mail info@biosigmaeu.com | http://www.biosigma.com |

M521D

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CE Marked product - manufactured in compliance with 93/42/EEC Directive Annex V Notified Body 0373 Istituto Superiore di Sanità - Roma

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DIRECTIONS FOR USE

Spatulas for cytological sampling without sharp edges.

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INSTRUCTIONS

Dilate the vaginal canal with the speculum and expose the cervix. Gently insert the tip of the spatula into the cervical canal and carefully rotate through 360° obtaining cells from ectocervix, transformation zone and lower endocervix. The spatula must make sustained contact with the cervix throughout the whole rotation so that no segments will be missed. Transfer the sampled cells to the slide. Fix the specimen immediately with fixative spray for cytology. Dispose the device.

Caution: Read the instructions before using the device. Do not reuse. Use only if the package is not damaged. Use immediately after opening. Dispose of after use according to the laws in force. For use only under trained medical staff.

Risk conditions

The only precaution to be observed is to avoid the presence of flames

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PACKAGING

The product is individually packed with transparent cellophane heat-welded to white medical paper one-sided glossy. Such an internal packaging guarantees an adequate waterproof protection and in this way our product may reach the end-user in safe and hygienic conditions. The article is individually packaged and sterilized by ethylene oxide.

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STERILIZATION/VALIDITY

In autoclave by ethylene oxide according to validated cycle.

Sterility guaranteed in unopened and undamaged package. Validity : 5 years

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STORAGE

Store in a dry place. Do not expose to direct sunlight or heat sources.

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DISCHARGE AFTER USE

The product is similar to hospital waste or hazardous waste that must be destroyed according to applicable regulations.